

MONOMATERIAL TEETH (KaVo Analog)

PRODUCT OVERVIEW

- Simple monomaterial teeth
- Dental hardness
- Tooth colored shade



INSTRUCTIONS FOR USE

When inserting the teeth into the jaws, push harder in the vertical direction. While inserting, observe the correct orientation of the nick on the stump and push inwards until it latches into place. Nick is oriented to the black line on the picture.



It is very helpful to hold the model reversed while inserting the teeth, where insertion of the tooth can be well controlled. You can also help yourself by pushing the tooth against a hard object (table).

RECCOMENDATION

We advise to use air/water extraction or water-cooling during preparation, as it is advised in the case of any other tooth preparation due to the removal of the preparation debris.

It is also recommended to use a mask or protective goggles during preparation.

We wish you many successfully practiced procedures !

Additional replicas can be ordered at www.biovoxel.tech.

