



BIOVOXEL
TECHNOLOGIES

BIOVOXEL TableClamp Basic




PRODUCT OVERVIEW

The TableClamp Basic is a practical table holder which allows you to hold the tooth in various angles.

You can work in direct view, indirect view or position the tooth in any angle needed.

The clamp includes:

- REMOVABLE RUBBER SLEEVE
- REMOVABLE TABLECLAMP HEAD
- DOUBLE JOINT
- LOCKING THUMBSCREWS
- TABLE VISE

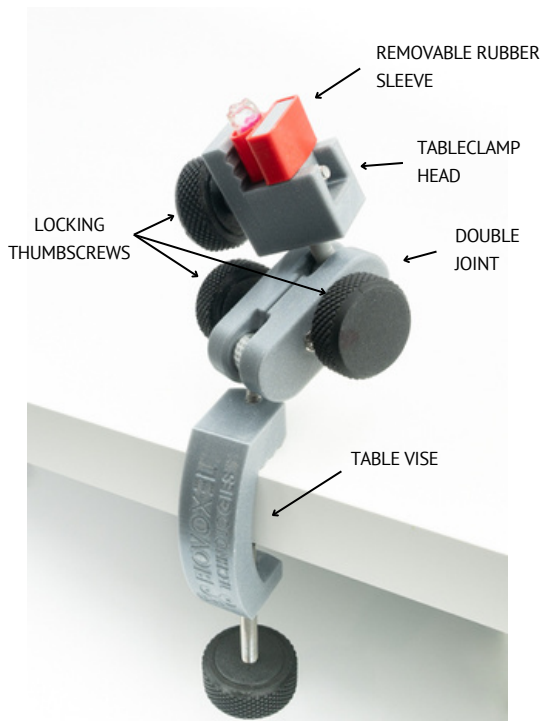

INSTRUCTIONS FOR USE


Before you start

- Attach the table vise to your table. (To prevent the damage to the table surface, use any cushioning material as a pad between the vise and the table).
- Fasten it securely to avoid instability. The table vise allows you to put the TableClamp on tabletops that are 0 - 4.5cm thick.
- Fasten both of the double joint thumbscrews.
- Fasten the whole TableClamp head (when the double joint thumbscrews are loosened, the head cannot be securely fastened).
- The TableClamp head is removable and can be exchanged for Biovoxel Typodont.


Adjusting the TableClamp head for practice

You can adjust the clamp according to your preference.

1. Loosen the thumbscrews on the double joint.
2. Hold the head in the desired position.
3. Tighten the thumbscrews on the double joint clockwise while holding the head.

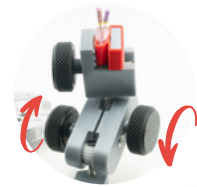

1.

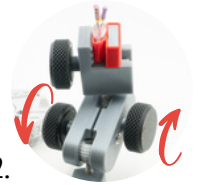

2.

Inserting the tooth

1. Place the tooth into the red rubber sleeve.
2. Fix the tooth by holding it and turning the thumbscrew clockwise.



Note: If you over-tighten the top lock thumbscrew, it is possible to fracture the tooth.

Apex Locator



If you wish to use apex locator together with the TableClamp Basic, insert the metal hook into the rubber sleeve and attach the file clip to the file (or work with endomotor directly).

The rubber sleeve needs to be filled with conductive liquid or gel (e.g. salty water).

Note: When working with Biovoxel Endoteeth, the apex locator starts reading values only after the whole canal has been cleaned of artificial pulp material and well irrigated.

Changing the head for Biovoxel Typodont Jaw

1. Make sure the double joint thumbscrews are well tightened.
2. Unscrew the the TableClamp head turning it counterclockwise.
3. Take out the head and replace it with the typodont jaw.
4. Screw the jaw clockwise until its tight in its place.



Spare parts

If you need to order spare parts or other products, please contact us at sales@biovoxel.tech or visit our web www.biovoxel.tech.



Feedback

If you have any questions or feedback towards further improvements of the model, please do not hesitate to contact us at info@biovoxel.tech or whatsapp.

Thank you for choosing our product holder.
We wish you a lot of quality training!



Biovoxel Technologies Team



BIOVOXEL
TECHNOLOGIES

www.biovoxel.tech