



BIOVOXEL
TECHNOLOGIES


Mandibular Anesthesia Model



PRODUCT OVERVIEW

Model includes:

- Replacable soft silicone gingiva
- The teeth with realistic anatomy and naturally shaped roots (from 38 to 48 in lower jaw)
- The jaw bone

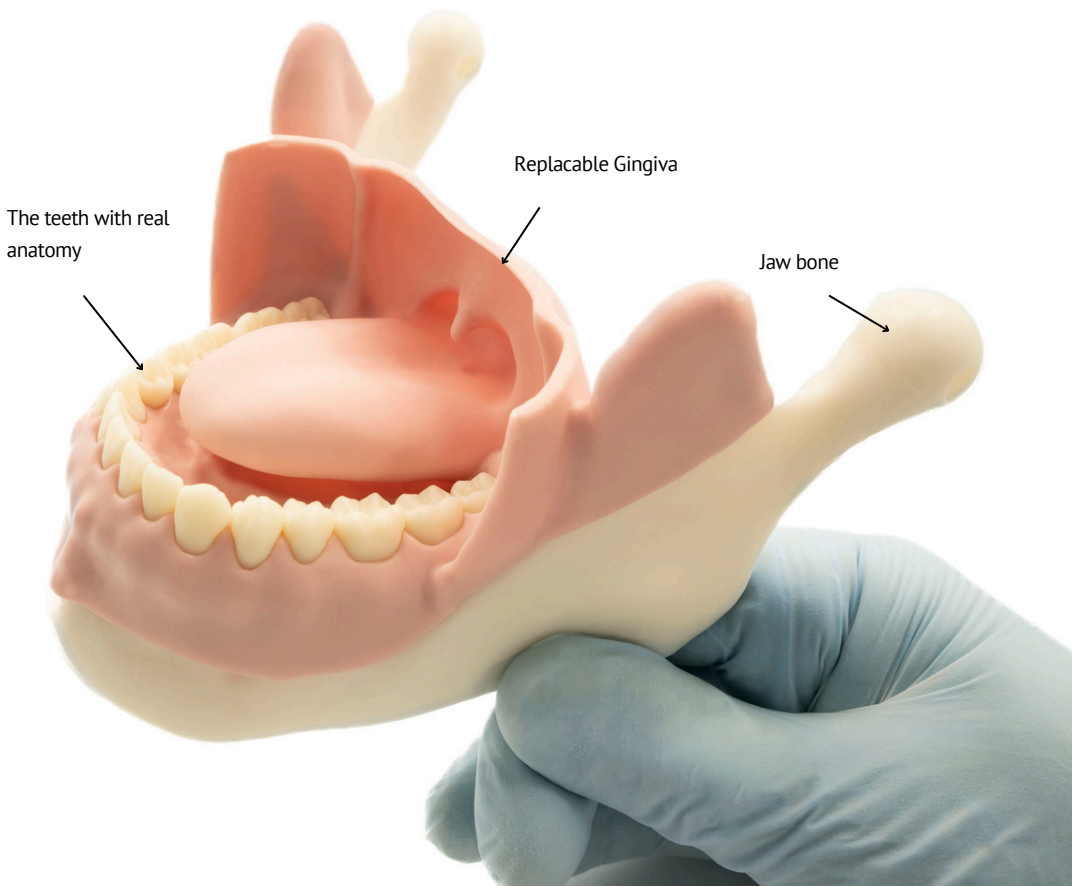

INSTRUCTIONS FOR USE

Anesthesia training

- Practice direct/ indirect mandibular nerve block

The model allows you to check your needle tip position from the dorsal side.

- Practice infiltration nerve block
- Practice intraligamental nerve block



Replacing the Gingiva

- 1/ Remove all the teeth from the model by pulling the teeth out by hand or gently with forceps.
- 2/ Remove the silicone gingiva.
- 3/ Place the new silicone gingiva onto the alveolus.
- 4/ Reattach the teeth.

Note: If the teeth are loose and fall out, we recommend inserting a small piece of adhesive gum to the root under the tooth, e.g. "PRITT Multi-Fix Sticky Dots"

Spare parts

If you need to order spare parts, please write us at sales@biovoxel.tech or visit our web.

Feedback

If you have any questions or feedback towards further improvements of the model, please do not hesitate to contact us at info@biovoxel.tech or whatsapp.

We wish you a lot of quality training!

Biovoxel Technologies Team



www.biovoxel.tech