

ENDOTOOTH



REPLICAS OF REAL TEETH FOR TRAINING IN ENDODONTICS

Utilization of high-end 3D printing technologies and microCT imaging

Transparent material with dentin hardness

Great variety available

Custom-made replicas on demand



WWW.BIOVOXEL.TECH



ENDOTOOTH



INSTRUCTIONS FOR USE

The teeth can be accessed using a high-speed handpiece. The root canals are suitable for preparation with both hand and rotary files. Ultrasonic tips can be used in a wet environment.

All methods of obturation are possible; when using thermocondensation methods, utilization of the lowest temperature is recommended due to material thermoplasticity.

Radiographs should be taken using a short exposure time and the lowest kV.

Material properties get enhanced when the tooth is soaked in water for 1 hour before use (artificial pulp gets softer).

Material simulating pulp tissue is suitable for chemical irrigation. For irrigation, use water, alcohol, or sodium hypochlorite solution. We recommend rinsing with irrigation as often as when working with real tooth.



WWW.BIOVOXEL.TECH