




**BIOVOXEL
TECHNOLOGIES**

BIOVOXEL ENDOPAN PRO

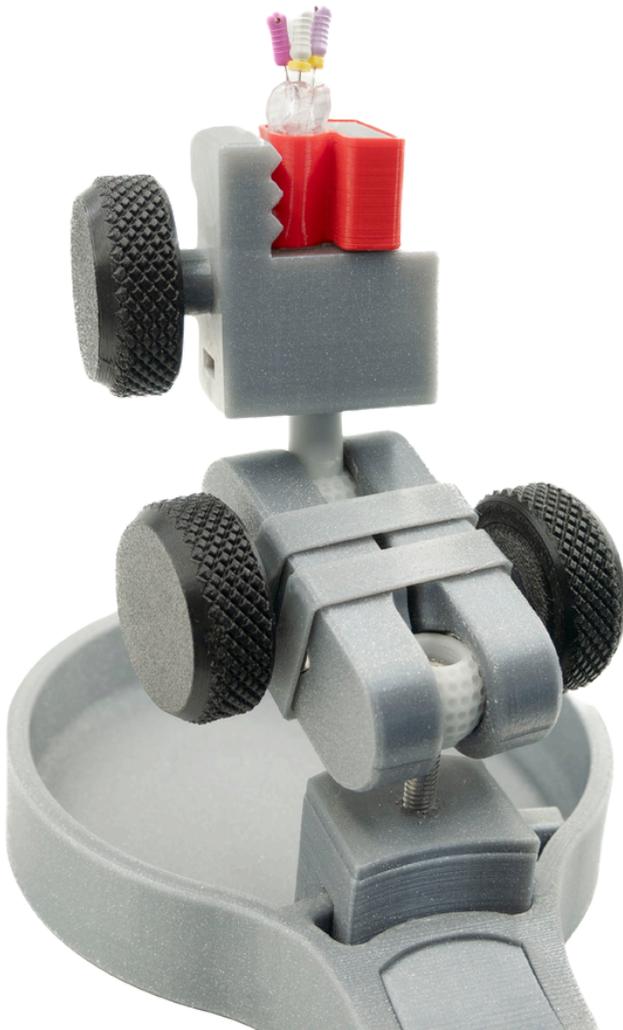


PRODUCT OVERVIEW

The EndoPan PRO is a practical table holder which allows you to hold the tooth in various angles.

You can work in direct view, indirect view or position the tooth in any angle needed.

The clamp includes:

- REMOVABLE RUBBER SLEEVE
- REMOVABLE HEAD
- DOUBLE JOINT
- LOCKING THUMBSCREWS
- CONTAINER FOR RINSING SOLUTION
- MOVABLE JOINT
- TABLE VISE

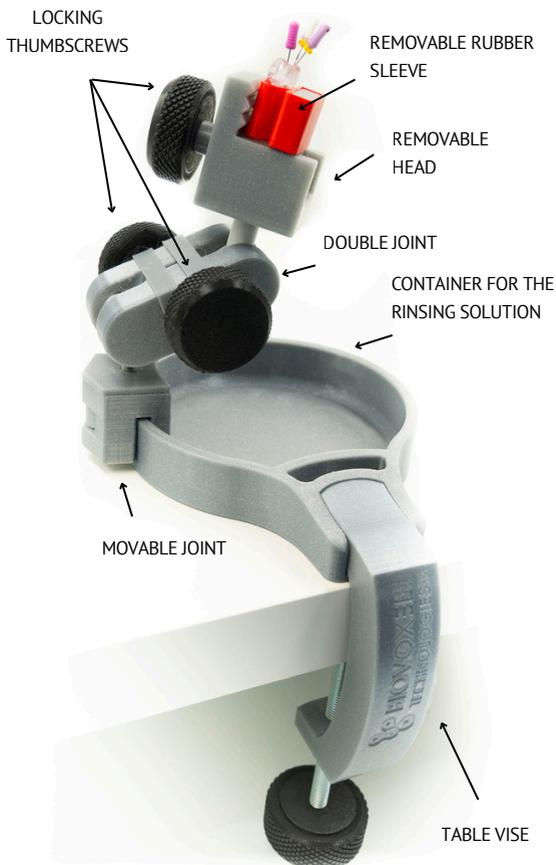

INSTRUCTIONS FOR USE

Before you start

- Attach the table vise to your table.
- Fasten it securely to avoid instability. The table vise allows you to put the EndoPan PRO on tabletops that are 0 - 7 cm thick.
- Fasten both of the double joint thumbscrews.
- Fasten the whole head (when the double joint thumbscrews are loosened, the head cannot be securely fastened).
- The EndoPan PRO's head is removable and can be exchanged for Biovoxel Typodont.

Adjusting the EndoPan PRO's head for practice

You can adjust the EndoPan removable head according to your preference.

1. Loosen the thumbscrews on the double joint.
2. Hold the head in the desired position.
3. Tighten the thumbscrews on the double joint clockwise while holding the head.



1.



2.

Inserting the tooth

1. Place the tooth into the red rubber sleeve.
2. Fix the tooth by holding it and turning the thumbscrew clockwise.



Note: If you over-tighten the top lock thumbscrew, it is possible to fracture the tooth.

Apex Locator

If you wish to use apex locator together with the EndoPan PRO, insert the metal hook into the rubber sleeve and attach the file clip to the file (or work with endomotor directly).

The rubber sleeve needs to be filled with conductive liquid or gel (e.g. salty water).

Note: When working with Biovoxel Endoteeth, the apex locator starts reading values only after the whole canal has been cleaned of artificial pulp material and well irrigated.



Changing the head for Biovoxel Typodont Jaw

1. Make sure the double joint thumbscrews are well tightened.
2. Unscrew the the head turning it counterclockwise.
3. Take out the head and replace it with the typodont jaw.
4. Screw the jaw clockwise until its tight in its place.



1.



2.

Important Notice

Do not clean or sterilize the product in a dishwasher. Exposure to high temperatures or strong detergents may cause damage or deformation of the model.

Spare parts

If you need to order spare parts or other products, please contact us at sales@biovoxel.tech or visit our web www.biovoxel.tech.



Feedback

If you have any questions or feedback towards further improvements of the model, please do not hesitate to contact us at info@biovoxel.tech or whatsapp.

Thank you for choosing our product holder.
We wish you a lot of quality training!



The *Biovoxel Technologies Team*



www.biovoxel.tech

