




**BIOVOXEL  
TECHNOLOGIES**

# **BIOVOXEL ENDOPAN BASIC**

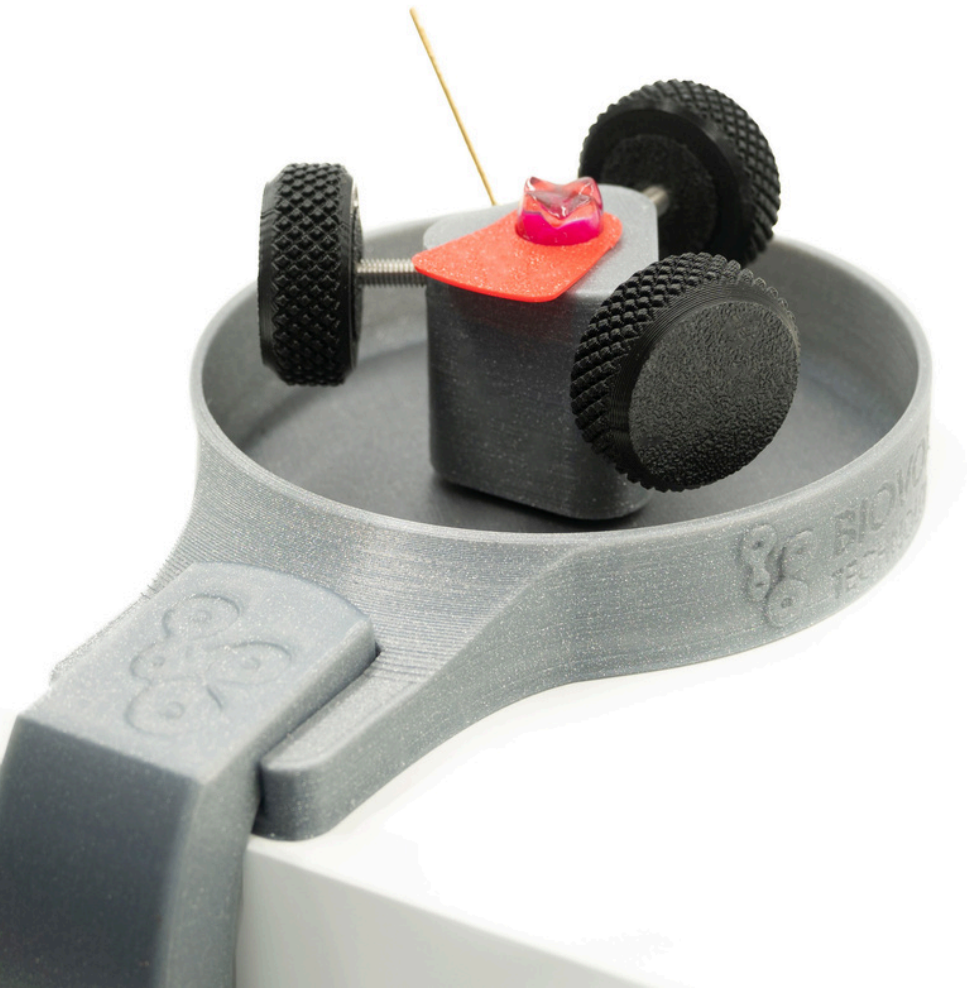


# PRODUCT OVERVIEW

EndoPan Basic is a practical and simple table holder for fixing any training tooth.

## The clamp includes:

- REMOVABLE RUBBER SLEEVE
- CONTAINER FOR THE RINSING SOLUTION
- LOCKING THUMBSCREWS
- TABLE VISE

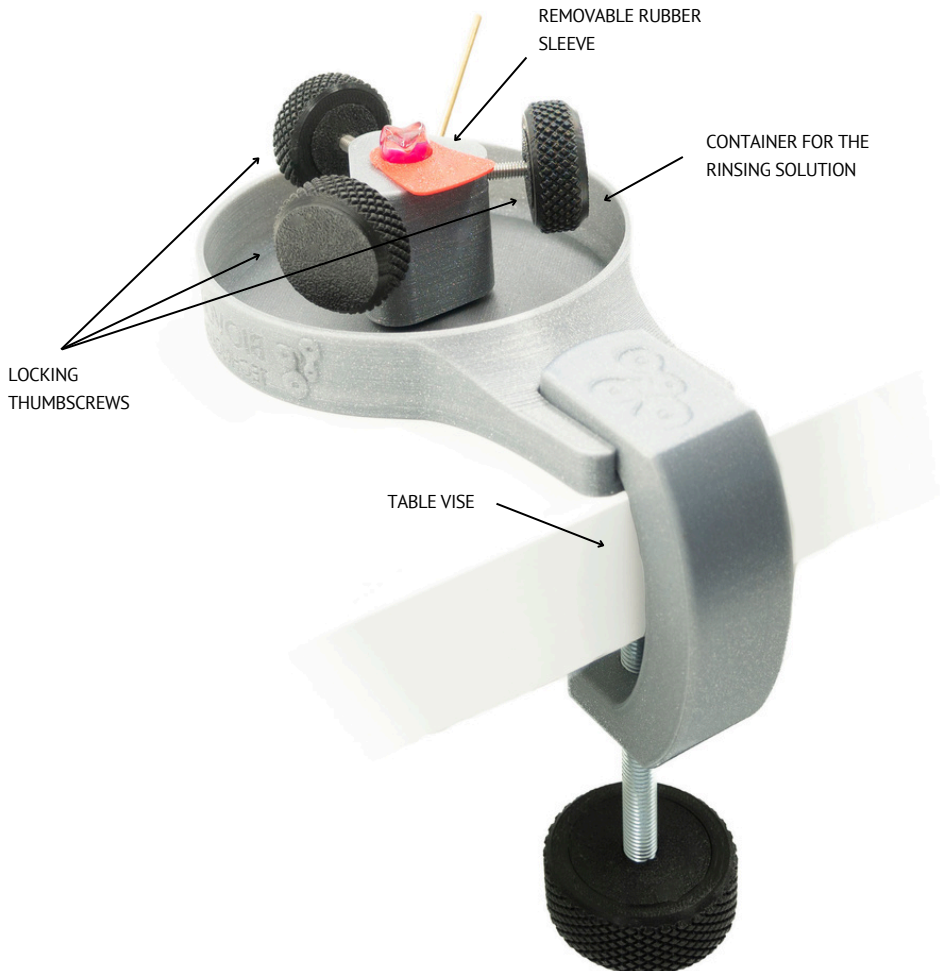

# INSTRUCTIONS FOR USE

## Before you start

- Attach the table vise to your table.
- Fasten it securely to avoid instability. The table vise allows you to put the TableClamp on tabletops that are 0 - 4.5cm thick.

## Inserting the tooth

1. Place the tooth into the red rubber sleeve.
2. Fix the tooth by holding it and turning the thumbscrew clockwise.

If you over-tighten the top lock thumbscrew, it is possible to fracture the tooth.



## Apex locator

If you wish to use apex locator together with the EndoPan Basic, insert the metal hook into the rubber sleeve and attach the file clip to the file (or work with endomotor directly).

The rubber sleeve needs to be filled with conductive liquid or gel (e.g. salty water).

Note: When working with Biovoxel Endoteeth, the apex locator starts reading values only after the whole canal has been cleaned of artificial pulp material and well irrigated.



## Spare parts

If you need to order spare parts or other products, please contact us at [sales@biovoxel.tech](mailto:sales@biovoxel.tech) or visit our web [www.biovoxel.tech](http://www.biovoxel.tech).

## Feedback

If you have any questions or feedback towards further improvements of the model, please do not hesitate to contact us at [info@biovoxel.tech](mailto:info@biovoxel.tech) or whatsapp.

Thank you for choosing our product holder.  
We wish you a lot of quality training!

*Biovoxel Technologies Team*

